NEDERLANDS

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Presentation: 20 - 50 gram tubes

Product description

Melladerm® Plus contains: honey, glycerin, propylene glycol and PEG 4000. When the gel is in contact with the wound, fluid is extracted from the surrounding tissues. The osmotic action creates a moist wound-healing environment. Honey together with the aforementioned moist wound healing environment stimulates the wound healing process, facilitates autolysis and promotes better epithelial cell migration. The gel will dilute gradually and can be removed easily. Melladerm® Plus is a primary wound dressing that can be covered by most commonly used secondary dressings.

Complete quantitative and qualitative composition

Melladerm® Plus consists of 45% honey. The composition below may differ slightly because honey is a natural product.

Nr.	component	%	min – max
1	Water	7,7	6,7 – 9
2	Fructose	17,5	12,5 – 18
3	Glucose	14	8,5 - 14,5
4	Maltose	3	2 – 4
5	Sucrose	1,8	0,05 – 2
6	PEG4000	16	0
7	Propylene glycol	34,5	0
8	Glycerol	4,50	0

Indications and Intended use

Superficial Chronic wounds - Decubitus, Ulcus Cruris, diabetic foot wounds, Fungating wounds (deodorising and debriding).

Contaminated Acute wounds – surgical wounds (postoperative wounds), Traumatic wounds (superficial wounds, cuts), Small burn and laser wounds (1° and 2° degree).

Melladerm Plus can be used both by professionals and laymen.

Contraindications

- No contraindications have been reported to date.
- Do not use the gel if the patient is highly prone to infection e.g. a patient with very large burn wounds.
- Do not use on patients with a known hypersensitivity to the gel or its components.
- Do not use on eyes or mucous membranes.
- Do not use if package is damaged or opened.

Side Effects

- Melladerm® Plus can in some cases cause a short stinging sensation.
- If a serious incident has occurred with regard to the medical device (Melladerm® Plus), the patient/user should report this to the manufacturer and the competent authority of the Member State where the user and/or patient.

Precautions and observations

- Do not use if the tube is damaged.
- If necessary consult a healthcare professional for the appropriate medical treatment.
- Inspect and clean the wound with a wound cleanser.
- Bacterial colonization of chronic wounds is always present and is not a

- contraindication for using Melladerm® Plus.
- Due to the autolytic debridement the wound may appear deeper after the first dressing changes.
- Seek medical supervision if signs of infection occur.
- Melladerm® Plus is for single patient use only, to avoid cross contamination.
- When the condition of the wound deteriorates, treatment with *Melladerm® Plus* should be stopped.
- Do not use the product continuously for more than 30 days.

Pregnancy, breastfeeding and use in Paediatric patients (0-15 years)

There is no information available about the use of Melladerm® Plus during pregnancy and breastfeeding. Are you pregnant or Breastfeeding? Please contact your doctor or pharmacist. No contraindications are known to date for the use of Melladerm® Plus in Paediatric patients.

Instructions for use

1. Clean the wound.
2. Remove lid and open tube in an aseptic way.
3. Squeeze the tube to extract the gel and apply it without touching the wound.
4. Use a (sterile) spatula when necessary.
5. Place a dressing on top of the gel to cover the wound.
6. Melladerm® Plus can remain on the wound up to 48 hours on average. Reapply the gel on the wound as soon as the gel has disappeared.

Remove

Melladerm® Plus should be changed when leakage of the secondary dressing occurs or when the secondary dressing needs to be removed. The ointment can be cleansed out of the wound with a sterile saline solution. Dressing change is depending on the wound fluid production of the wound, but is usual between 2 -7 days.

Waste processing/disposal

Cardboard box, paper leaflet for paper waste and the plastic tube for recyclable waste.

Storage precautions

- Store between 5°C - 25° C
- After opening the package, the product can be used for 28 days.

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© SanoMed Manufacturing BV 2022 Barring Spelling errors, SanoMed Manufacturing BV shall at all times retain the right to change its products and assortment.